Medrull®

RUBBER

HOT WATER bottle

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GB

WARNING! HOT WATER BOTTLES CAN CAUSE BURNS! AVOID PROLONGED DIRECT CONTACT WITH THE SKIN! DO NOT USE BOILING WATER! DO NOT OVERFILL! CHILDREN AND PERSONS WITH SPECIAL NEEDS SHOULD BE SUPERVISED WHEN USING A HOT WATER BOTTLE. PLASTIC BAGS CAN BE DANGEROUS. TO AVOID DANGER OF SUFFOCATION, KEEP THE PLASTIC PACKAGE AWAY FROM CHILDREN!

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LOT

DSTU 2667-94

Medrull® is a registered Trade Mark of FORANS International AG, Switzerland

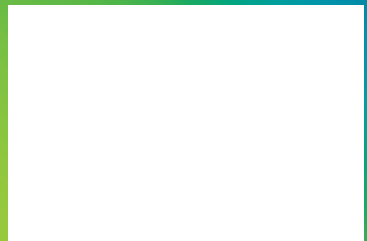
KIEVGUMA LLC, Ukraine, 07403, Kiev region, Brovary, 127 Kutuzova str.
Phone: +38(044)364-56-72; +38(044)364-56-74. E-mail: info@kievguma.ua

GB Importer/ FI Maahantuoja/ SE Importör:

FORANS Eesti AS, Posti 23, 74805 Loksa, Estonia, tpuh.: +372 6776426; e-mail: info@forans.ee

GB Distributor/ FI Jakelija/ SE Distributör:

Nutrielixir Oy, Pertunpellonraitti 2 O LTS, 00740 Helsinki, Finland. Puh. +358 (0)40 132 1115.
E-mail: info@nutrielixir.fi.

GB INSTRUCTION FOR USE

RUBBER HOT WATER BOTTLE

Type: A1, A2, A3.

Complete set: Hot water bottle from natural rubber with sleeve and stopper.

Indications: A hot water bottle is used for body warming, for individual application.

Conditions of use:

1. Check the bottle and stopper for integrity before use.
2. Wash the bottle with warm soapy water, then rinse with running water.
3. Fill the bottle with water not hotter than 50°C to 2/3 of capacity. Hold the bottle by the neck in an upright position and fill slowly to avoid hot water splashing back.

WARNING! DO NOT USE BOILING WATER. DO NOT OVERFILL AS THIS MIGHT CAUSE THE BOTTLE TO BURST. DO NOT FILL THE BOTTLE WITH HOT TAP WATER, WHICH CAN CAUSE THE RUBBER TO PERISH.

4. Screw the stopper tight.
5. Hold the bottle upside down over stopper, away from the body, and squeeze lightly to check for leaks.
6. Wipe the bottle dry.
7. It is necessary to enwrap the bottle with strong fabric or towel to avoid direct contact with the skin.
- PRECAUTION:** It is not recommended to sit or lie on top of the bottle and put on it heavy things.
8. For the avoidance of pigmentation, reddening (especially in the permanent, long-term application of the bottle) might be necessary to apply a moisturizing cream on the skin.
9. It is necessary to take care of the correct position of the bottle and check often the degree of skin warming as burns can occur. The degree of warming of the skin under the bottle is determined with the hand.
10. Empty the bottle after use, hangs it upside down, leaving with stopper removed until it dries completely.

Storage: Store at a temperature of 0°C to 25°C and relative humidity not more than 80%, at a distance of at least 1 m from heating appliances.

PRECAUTION: Keep away from direct sunlight. Prevent contact with different oils, solvents and greases. With intensive use, recommend replacement the product, every two years.

WARNING! PLASTIC BAGS CAN BE DANGEROUS. TO AVOID DANGER OF SUFFOCATION, KEEP THE PLASTIC PACKAGE AWAY FROM CHILDREN.

Disposal: Dispose of the appliance at the end of its life span according to the local recycling regulations. IN CASE OF SERIOUS INCIDENTS, REPORT TO THE MANUFACTURER AND COMPETENT AUTHORITY.

RETAIN THESE INSTRUCTIONS FOR FUTURE REFERENCE.

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