THC

PÄIHDEPIKATESTI

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Salcare®
RAPID SUBSTANCE-ABUSE HOME TEST
for detecting the use of cannabis products

INSTRUCTIONS FOR USE

Read these instructions for use carefully before performing the test and keep these instructions for use for future reference.

MANUFACTURED AND MARKETED BY

SALOFA OY
Satamakatu 16, 24100 SALO
info@salofa.com
www.salofa.com

INTENDED USE

This Salcare® substance abuse test is designed to detect the use of cannabis products in human urine. The test is designed to distinguish a negative result from a positive result.

The test does not indicate the concentration of the test substance, only the presence in the sample. The test is not an in vitro diagnostic (IVD) medical device and is not intended for medical purposes.

TESTED SUBSTANCE

Tetrahydrocannabinol (THC) THC is the primary intoxicating ingredient in cannabis.

It is important to consider the following regarding the rapid substance-abuse test:

It is possible that an error during the test or other substances in the urine sample, may interfere with the test and cause false-positive results.

PACKAGE CONTENTS AND OTHER NECESSARY EQUIPMENT

The product package contains:

- 5 test strips in foil pouches.
In addition to the test cartridge, the foil pouch contains a desiccant packet.
- Instructions for use.

Other equipment needed to perform the test not included in the package:

- Dry and clean container for the urine sample.
- Clock or timer for timing.


PRODUCT STORAGE AND DISPOSAL

- Store at refrigerator temperature or at room temperature (+2 – +30 °C).
- Keep out of the reach and sight of children.
- The product expiration date is indicated on the outer packaging and on the foil pouch of the test strip.
- Used or expired products and packaging must be disposed of in accordance with waste management regulations. The product must not be disposed of in the environment.
Cardboard packaging and instructions: energy waste.
Foil pouch, test strip, and desiccant packet: mixed waste.

CONSIDERATIONS / PRECAUTIONS

- Do not reuse as the product is for single use only.
- The foil pouch of the test strip must not be opened before the test is carried out.
- When handling the test strip, hold the end of the strip marked with the tested substance.
Do not touch other parts of the test strip to avoid damage and contamination of the test strip.
See figure: 'Parts of the test strip' on page 14.
- During and after the test, ensure hand hygiene and the proper cleaning or disposal of materials in contact with the urine sample.
- Do not use the test after the expiry date indicated on the product.
- Do not use the test strip if its foil pouch is damaged.
- Consider the completion time of the test (5–10 min) before interpreting the results.

PARTS OF THE TEST STRIP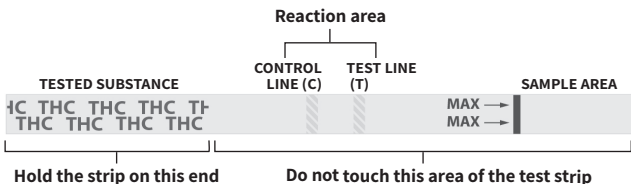

PERFORMING THE TEST

Make sure that the test strip is at room temperature before performing the test.

Collect a urine sample in a dry and clean container. The urine sample must be collected so that testing can be performed as soon as possible after collection.




1. Open the foil pouch and remove the test strip from the pouch.

N.B! Do not open the foil pouch until you are ready to perform the test.

2. Hold the sample area of the test strip in the urine sample for approx. 30 seconds. After sampling, place the test strip on a hard, clean surface. An absorbent surface may interfere with the operation of the test. Wait 5 minutes.

3. Read the results within 5-10 minutes of performing the test.

The result is not reliably readable after 10 minutes.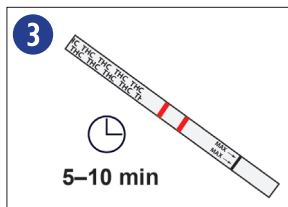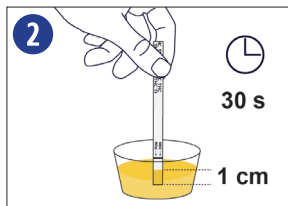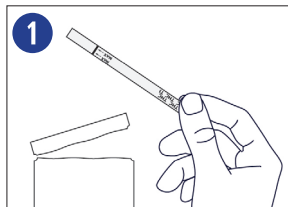


INTERPRETATION OF RESULTS

Read the results within 5–10 minutes of performing the test.

The test result is interpreted based on the coloured lines formed in the reaction area. The colour intensity of the lines may vary.

After the test, a control line should be visible in the reaction area at C. In this case, it can be confirmed that the test has been performed correctly.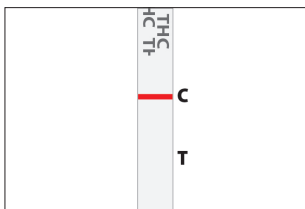


POSITIVE RESULT

Test substance detected in the sample

One coloured line appears in the control line area (C).


No line appears in the test line area (T).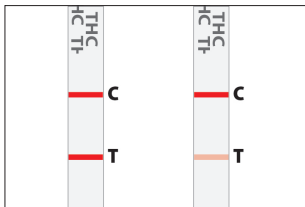

NEGATIVE RESULT

No test substance detected in the sample

Two coloured lines appear in the reaction area, one in the control line area (C) and the other in the test line area (T).

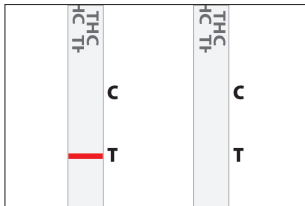FAILED RESULT

The result cannot be interpreted

If no control line (C) appears in the reaction area, the result cannot be interpreted.

To obtain an acceptable result, the test should be repeated with a new test strip.

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